

# TONELINK™ APP

QUICK GUIDE



FOR ALL WIDEX MOMENT™ MODELS

**WIDEX**

SOUND LIKE NO OTHER



# GETTING STARTED

The **TONELINK** app is designed to work with all **WIDEX MOMENT** hearing aids and can connect to Apple and Android phones.

Download the **TONELINK** app from the App Store or Google Play.



# HOW IT WORKS

The **TONELINK** app uses your phone's speakers to send instructions to your hearing aids using high-frequency sounds.


The first time you use the app, you have to pair your hearing aids with your phone. Follow these steps on the next page and note that you may have to adjust your phone's volume in the process.

# PAIRING TONELINK WITH YOUR PHONE

1. Open the **TONELINK** app and follow the instructions on the screen
2. Find and scan the QR code in the hearing aid user guide, if you were given one by your hearing care professional. If not, press **Skip**
3. Restart your hearing aids by turning them off and then on again and then put them on
4. Pair your hearing aids with your phone by pressing **Pair**
5. If you hear a sound in your hearing aids, press **Yes**
6. If you don't hear a sound in your hearing aids, press **No**. Next, adjust the volume on your phone and press **Pair** again

The **TONELINK** app helps you switch between programs. By scanning the QR code from the hearing aid user guide, the app can display the programs that are specifically chosen for you.

# TONELINK APP HOME SCREEN

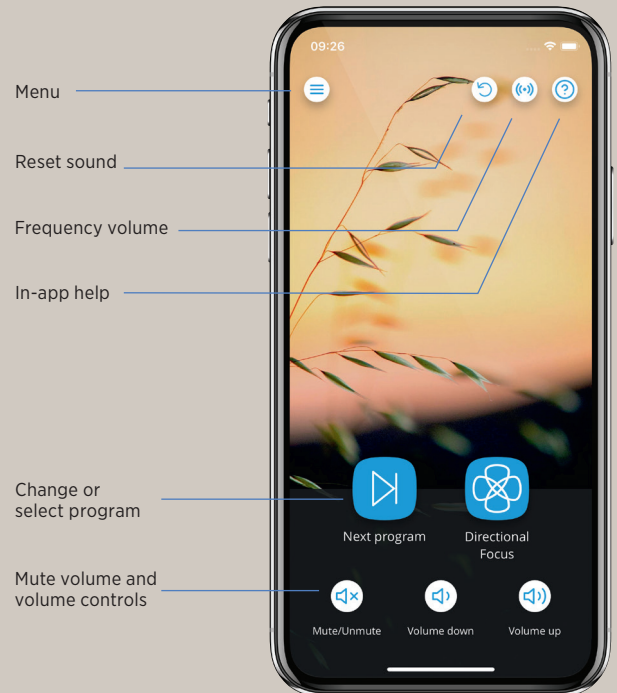

# EASY CONTROL OF KEY FUNCTIONS

With **TONELINK** you can easily:

- Change programs
- Adjust volume
- Mute
- Change directional focus

## GET MORE INFORMATION

[widex.com/tonelink-app](https://www.widex.com/tonelink-app)

## IS YOUR PHONE COMPATIBLE?

[www.global.widex.com/compatibility](https://www.global.widex.com/compatibility)

## WANT TIPS OR WANT TO LEARN MORE ABOUT HEARING AND HEARING LOSS?

<https://www.widex.com/en-us/blog>





SOUND LIKE NO OTHER



Printed 2021-03. 9 502 5402 041 #01 / CBR04M/0321

Widex is a registered trademark of Widex A/S.

WIDEX A/S, Nymøllevej 6, DK-3540 Lyngø

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

Android, Google Play, and Google Play logo are trademarks of Google Inc.

Subject to change with no further notice.